



"C" SPLINT SOFT

Cleaning and Maintaining

1. DAILY CLEANING

Greater "C" recommends cleaning your appliance daily before and after every use. Immediately after removal, use warm water (not to exceed 45°C/113°F) and soap to remove remaining saliva and debris from the device.

2. USING CLEANING AGENTS

For periodic removal of debris and maintenance, you can use any mild over-the-counter cleaning agents designed for oral devices. If needed, use a soft bristle brush to clean the inside of your appliance for debris removal. Some of the validated cleaning agents include Dawn dish soap, Polident, Efferdent, 3% peroxide solution, 15% bleach solution, and mouthwash.

3. CAUTIONS

- DO NOT SOAK your appliance in any liquid for more than 1 hour.
- DO NOT use abrasive cleaning agents (i.e. toothpaste), as it can shorten the life of your appliance.
- Dispose of the cleaning solution after every use. Rinse the appliance well prior to use.
- DO NOT SOAK the appliance in water hotter than 45°C/113°F due to risk of warping the device.
- Ensure to dry your appliance prior to storing it in the case to prevent bacterial growth.