



# CU-SIL DENTURE

*Cleaning and Maintaining*

## 1. DAILY CLEANING

Greater "C" recommends using a Cu-Sil cleanser daily to preserve and protect your denture.

## 2. USING CLEANING AGENTS

The Cu-Sil cleanser will help protect and prolong the life of your denture. Use daily and for heavier stains, Kleenite denture cleanser may be safely used. In the event that heavy calculus or tartar accumulate due to irregular daily cleaning, Stain Away Plus may be safely used.

## 3. CAUTIONS

- DO NOT use denture tablets, as they can cause bleaching and deterioration, which will shorten the life of your Cu-Sil denture.

## 4. MAINTAIN

Contact your dental professional if you experience any damage or discomfort with your Cu-Sil denture. Your dental professional will advise you when you come in for a regular checkup.